## **Printing DICOM images from a folder or from imported media**

The program is fully functional for 15 days, after which registration is required for further use. Contact email: [digitalidx@gmail.com](mailto:digitalidx@gmail.com).

The program is compatible with all types of DICOM viewer software that have the ability to export specific DICOM images to a desired folder or from the provided media.

It's recommended to first select the necessary series of characteristic images during the review. The images are automatically loaded by opening the respective folder.

General parameters required for communication with the printer, the location of the image folder, and the number of divisions per film are entered in an ini file alongside the program.

The program consists of several grouped segments of functions that are easy to understand. You can load the entire study, but this will take a long time and unnecessary resources. At the bottom, there's a progress bar where you can track the loading process by changing the page number. You can forcibly interrupt the program at any time and then restart the loading.

On the left side, from top to bottom, there are buttons with specific processing functions for the selected image. A specific image in the grid is selected with a left click, and it will be highlighted with a green frame. Then, specific functions are applied to that image. Click "Next" to apply effects, send a single image, or send all pages to print in the upper right corner.

Deselecting a chosen function is done by right-clicking anywhere on the black area, which changes the cursor back to an arrow. You can select multiple images in the grid by holding down the CTRL key on the keyboard and clicking on the images in the grid. Choose the desired tool, and the effect will be applied to all selected images.

Depending on the number of loaded images, you might have multiple film pages like PAGE1/2. "Next Page" displays the following page, and "Prior Page" displays the previous page. Each page can be individually adjusted based on the number of images. Click the Grid field to open squares. Hold the left mouse button and move within the division to increase the required number of images, or select 2x2 squares for 4 images.

If you press the space bar on the keyboard, you'll access a menu with additional tools that might be of interest to you.

For more information contact +381 64 640 8812, e-mail: [digitalidx@gmail.com](mailto:digitalidx@gmail.com)

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